


### AI Dermatology Left and Right Ears Sample Preparation Guide

- Prepare the slide by rolling the swab across the slide, ensure to avoid rubbing the swab against the slide to prevent cell damage.
- Orient the frosted edge to the right side when placing prepped, stained and cover slipped slide on the Imagyst scanner.

**Horizontal Orientation** – Roll samples across the slide from non-frosted edge to frosted edge (arrow shown) with left ear swab on top and right ear swab on the bottom.

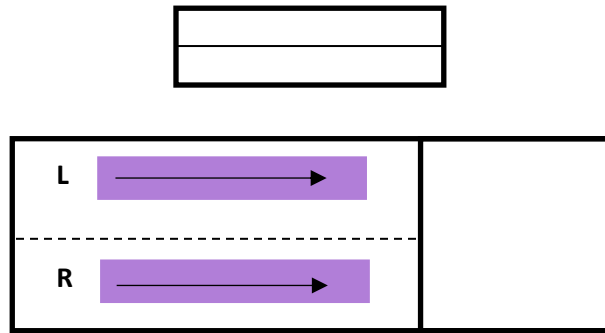

**Vertical Orientation** - Divide the slide in half. Create the left ear sample by rolling the swab in two short horizontal parallel lines from the non-frosted edge to the middle of the slide (arrow shown). Create the right ear sample by rolling the swab in two more short horizontal parallel lines starting from the middle of the slide to the frosted edge (arrow shown).

