
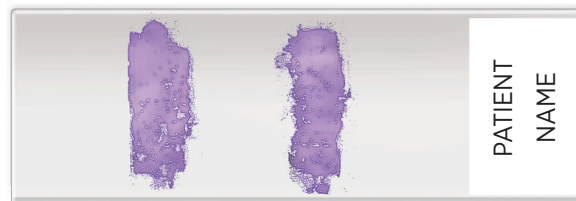


vetscan Imagyst®


AI Dermatology

Left and Right Ears Sample Preparation Guide

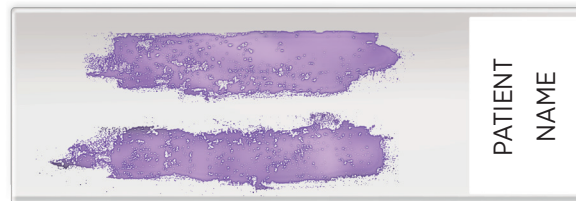

	Label left and right ears horizontally or vertically
	Gently roll the swab across the slide
	To a stained slide, add 1 to 2 drops of immersion oil
	Place 24 x 60 blank coverslip
	Check and adjust scan area to cover cellular content

	Rub the swab against the slide
	Create thick sample areas
	Use too much immersion oil
	Use heat to fix
	Select white space in scan area

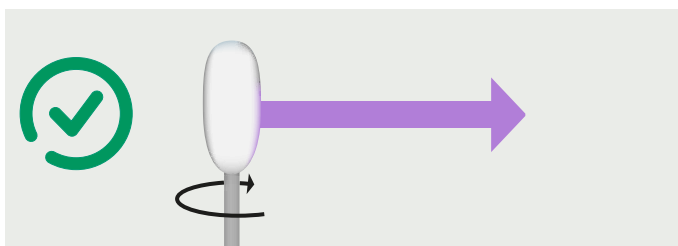

Use the stains according to manufacturer's guidelines.
Change the stains according to industry best practices.


vetscan Imagyst®

AI Dermatology

Non-Ear Sample Preparation Guide



Skin Swab

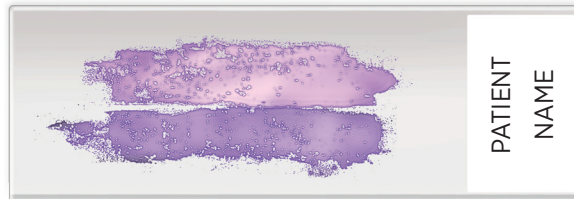

Impression Smear



DO



DON'T



Label the slide



Skin swab – Gently roll the swab across the slide in parallel lines



Impression Smear – Press slide into lesion



To a stained slide, add 1 to 2 drops of immersion oil



Place 24 x 60 blank coverslip



Check and adjust scan area to cover cellular content



Skin Swab – Rub the swab back and forth



Impression Smear – Rub the slide back and forth across lesion



Create thick sample areas





Use too much immersion oil



Use heat to fix



Select white space in scan area

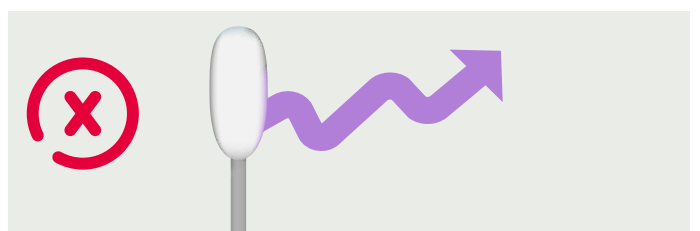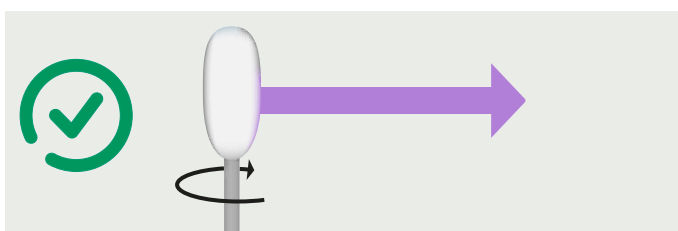

Use the stains according to manufacturer's guidelines.
Change the stains according to industry best practices.